



PACKING LIST

1 - Activated Carbon Filter with Gasket

ABOUT THE PRODUCT

This filter is designed to make your job safer and more comfortable by removing most odors, fumes and vapors from the air processed by the Vanguard Mobile 3. This filter is designed to be used internally along with the HEPA Filter of your Vanguard Mobile 3.

INSTALLATION INSTRUCTIONS

1. Turn off the unit and remove the power cord.
2. Wearing full PPE (per the CDCs recommendations), set aside a sealable bag for disposing the filter(s).
3. Remove the filter access panel from the unit by unscrewing the four black thumb screws on the front of the unit and set aside for later use. The Activated Carbon Filter and HEPA Filter should now be exposed.
4. Slide out the existing Activated Carbon Filter and place the filter into a sealable bag for proper disposal. If necessary, you may need to slide out the HEPA Filter located beneath the Activated Carbon Filter to facilitate easier removal of the Activated Carbon Filter.
5. Leave the filter(s) in a closed bag after removal for 3 days, then dispose in the trash.
6. Remove all packaging from the replacement Activated Carbon Filter. Carefully align the Activated Carbon Filter with the outer walls of the unit, sit the filter onto the HEPA Filter and slide in until the filter stops.
7. Remount the filter access panel back on the unit and secure with the four thumb screws.
8. Replace the Activated Carbon Filter after 15 hours of use; remove filter and store in a closed plastic bag if time between unit usage exceeds 4 days.